



# How to use **Axiostat** in Vascular Procedure

## **Femoral** Access


### Using Manual Compression

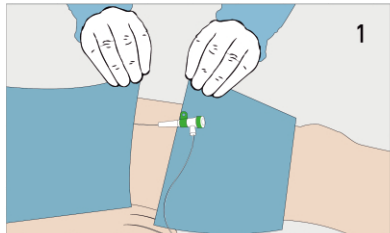


1 Remove the procedure coverings and clean the area around the catheter.

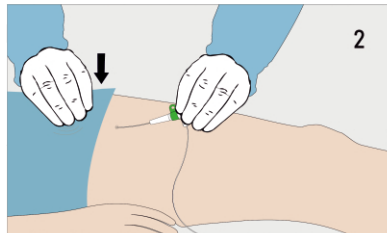


2 Apply proximal pressure 2-3 cm from sheath and remove the sheath and hold proximal pressure to stop the blood flow.

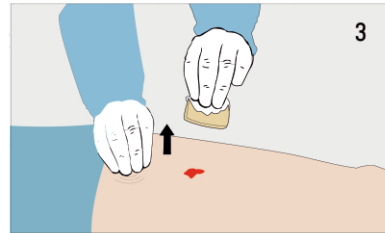


3 Release a little proximal pressure and allow 2-3 drops of blood and then apply Axiostat on the puncture site over the 2-3 drops of blood and make sure blood enters Axiostat to activate it.

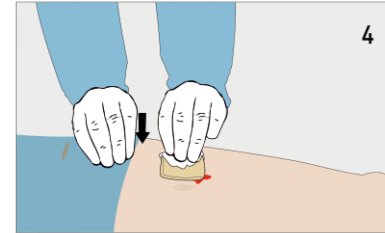


4 Apply pressure on proximal site and puncture site covered with Axiostat V55 immediately for 3 minutes.

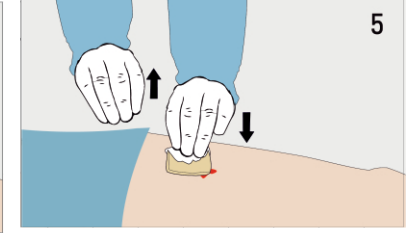


5 After 3 minutes, gradually release the proximal pressure and continue applying pressure only onto the puncture site for another minute (Puncture site 4 min full pressure).

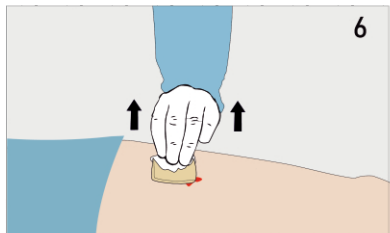


6 Gradual release pressure after 4 minutes till 5 minutes and make sure Axiostat adheres to the puncture site & also check for any signs of oozing from the puncture site.

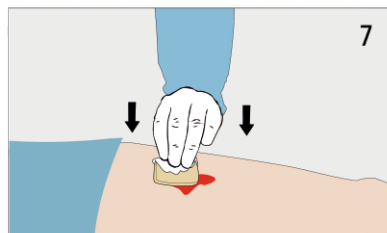

7 If oozing continues, hold the pressure onto the puncture site for 2-5 more minutes.

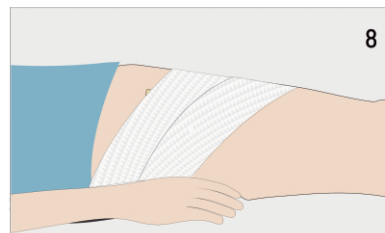

8 If there is no oozing and Axiostat is adhered to the site, use a gauze pack along with a secondary compression dressing to hold a constant pressure onto the puncture site for 6 hours - 24 hours without any sandbag or additional pressure over puncture site.

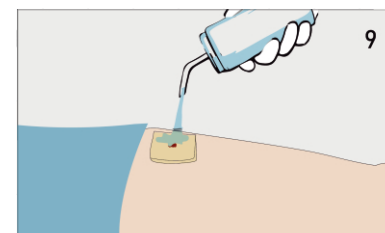

9 While removal, irrigate the Axiostat dressing with water or saline and gently remove it.

**Disclaimer:** Refer to the product labels and package insert for complete warnings, precautions, potential complications, and instructions for use.



**AXIO BIOSOLUTIONS PRIVATE LIMITED**

Plot 18, Gujarat Pharma Techno Park, Sari Matoda, Sanand Taluka,

Ahmedabad - 382220, Gujarat, India

Customer care: +91 8860786067, Email: [info@axiobio.com](mailto:info@axiobio.com) | [www.axiobio.com](http://www.axiobio.com)

[twitter.com/axiobio](https://twitter.com/axiobio) [facebook.com/axiobio](https://facebook.com/axiobio)

**axiostat**<sup>®</sup>  
stops bleeding, instantly

Distributed in Singapore by:

**Intellimed  
Healthcare**

Please contact +65 8218 3535, +65 9322 2803 for more information.